English

# Reusable Spider™ EEG [Electroencephalography] Electrodes

## Intended Applications

EEG, EP [Evoked Potentials], ICU [Intensive Care Unit], LTME [Long Term Monitoring], PSG [Polysomnography], IONM [Intraoperative Neurophysiological Monitoring], and Ambulatory.

## Sterilization

Non-sterile.

## Caution

Reusable Spider EEG Electrodes are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product.

## Packaging

Ten Electrodes per pouch.

## Instructions for Use

Select appropriate Reusable Spider Electrode type and size. Clean application site. Apply Electrode using conductive gel, paste, collodion or other similar adhesive. When finished, remove Electrode and clean the application site.

## Cleaning and Disinfecting Instructions

Follow acceptable cleaning and disinfecting protocols, including - but not limited to - washing with water and using disinfecting products. If skin was treated with an abrasive agent during application process, disinfect according to the proper protocols. Do not reuse Electrodes that have been contaminated with blood. Do not sterilize.

*This product has a useful shelf life of 5 years from the date of manufacture. Reusable product should be replaced if signal degrades.*

 RhythmLink International, LLC  
1140 First Street South  
Columbia, SC, USA 29209-3540  
+1.866.633.3754 [toll-free]  
+1.803.252.1222  
+1.803.252.1111 [fax]  
sales@rhythmink.com  
Rhythmink.com

**European Authorized Representative**  
Mdi Europa GmbH  
Langenhagener Str.71  
D-30855 Hannover-Langenhagen

**FOR SINGLE USE ONLY**  
**ASSEMBLED IN CHINA**

**Caution:** Federal Law [USA] restricts this device for sale by or on the order of a licensed medical practitioner.

RhythmLink® is a registered trademark of RhythmLink International, LLC. Spider™ is a trademark of RhythmLink International, LLC.

RhythmLink is the exclusive licensee of Spes Medica for the Spider Electrode.

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