English

Disposable Sticky Pad™ Surface Electrode

Intended Applications

EEG [Electroencephalography], EP [Evoked Potentials], IONM [Intraoperative Neurophysiological Monitoring], ICU [Intensive Care Unit], NCS [Nerve Conduction Studies], LTM [Long Term Monitoring], PSG [Polysomnography] and Ambulatory.

Caution

Sticky Pad Electrodes are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product.

Storage

Keep the package away from sunlight. Store at 10-40°C [50-104°F].

Instructions for Use

Select appropriate Sticky Pad Electrode. Remove excess hair, oils and dirt. Prep application site, remove Mylar backing from Electrode and place on application site. Apply pressure to center of Electrode and move to edges. Remove by pulling directly on Electrode. **DO NOT PULL ON THE CABLE!** Remove remaining hydrogel with clean, soapy water. This product is single-patient use only. Discard Electrode after use.

RhythmLink International, LLC
1140 First Street South
Columbia, SC, USA 29209-3540
+1.866.633.3754 [toll-free]
+1.803.252.1222
+1.803.252.1111 [fax]
sales@rhythmLink.com
RhythmLink.com

FOR SINGLE USE ONLY

EC REP
European Authorized Representative
Mdi Europa GmbH
Langenhagener Str.71
D-30855 Hannover-Langenhagen

To view a list of symbol definitions found on packaging and instructions for use, please visit RhythmLink.com/symbols.

RhythmLink® is a registered trademark of RhythmLink International, LLC.
Sticky Pad™ is a trademark of RhythmLink International, LLC.

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Rhythmink® est une marque déposée de Rhythmink International, LLC.
Sticky Pad™ est une marque déposée de Rhythmink International, LLC.
Rhythmink® è un marchio registrato di Rhythmink International, LLC.
Sticky Pad™ è un marchio registrato di Rhythmink International, LLC.
Rhythmink® ist eine eingetragene Marke von Rhythmink International, LLC.
Sticky Pad™ ist eine eingetragene Marke von Rhythmink International, LLC.