English

## Disposable EEG [Electroencephalography] Cup and Webb\* Electrodes

### Intended Use

RhythmLink Surface Electrodes are intended for non-invasive use with recording and monitoring equipment, (active and reference), of Electromyography (EMG), Electroencephalography (EEG) and Evoked Potentials.

### Intended Applications

EEG, EP [Evoked Potentials], IONM [Intraoperative Neurophysiological Monitoring], ICU [Intensive Care Unit], LTM [Long Term Monitoring], PSG [Polysomnography] and Ambulatory.

### Caution

Disposable EEG Surface Electrodes are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product.

### Instructions for Use

Clean application site. Apply electrode using conductive gel, paste, colloidion or other similar adhesive. When finished, remove electrode and clean application site. This product is single-patient use only. Discard electrode after use according to hospital protocols for biohazardous materials.

Avoid prolonged or repeated exposure to substances containing acetone or ethyl acetate. These solvents can damage the electrode and may lead to premature product failure.

The device is not made with natural rubber latex and nickel.

Reuse of this device may cause cross contamination and has NOT been validated for multiple uses.

L175001 rev010

RhythmLink International, LLC  
1140 First Street South  
Columbia, SC, USA 29209-3540  
+1.866.633.3754 [toll-free]  
+1.803.252.1222  
+1.803.252.1111 [fax]  
sales@rhythmLink.com  
RhythmLink.com

To view a list of symbol definitions found on packaging and instructions for use, please visit [RhythmLink.com/symbols](http://RhythmLink.com/symbols).

RhythmLink® is a registered trademark of RhythmLink International, LLC.

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RhythmLink® é uma marca comercial registrada da RhythmLink International, LLC.