English

Disposable Webbed* EEG [Electroencephalography] Electrode

Intended Applications

EEG, EP [Evoked Potentials], IONM [Intraoperative Neurophysiological Monitoring], ICU [Intensive Care Unit], LTM [Long Term Monitoring], PSG [Polysomnography] and Ambulatory.

Caution

Disposable Webbed EEG Electrodes are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product.

Instructions for Use

Clean application site. Apply Electrode using conductive gel, paste, collodion or other similar adhesive. When finished, remove Electrode and clean application site. This product is single-patient use only. Discard Electrode after use.

Avoid prolonged or repeated exposure to substances containing acetone or ethyl acetate. These solvents can damage the electrode and may lead to premature product failure.

RhythmLink International, LLC
1140 First Street South
Columbia, SC, USA 29209-3540
+1.866.633.3754 [toll-free]
+1.803.252.1222
+1.803.252.1111 [fax]
sales@rhythmLink.com
RhythmLink.com

FOR SINGLE USE ONLY

Australian Sponsor
Emergo Australia
Level 20 Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

To view a list of symbol definitions found on packaging and instructions for use, please visit RhythmLink.com/symbols.

RhythmLink® is a registered trademark of RhythmLink International, LLC.
*US Patent No. D 644,738

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Rhythmink® est une marque déposée de Rhythmink International, LLC.
Rhythmink® è un marchio registrato di Rhythmink International, LLC.
Rhythmink® ist eine eingetragene Marke von Rhythmink International, LLC.