English

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Français

Disposable Monopolar Stimulating Probe with Removable Handle

Intended Use

The RhythmLink International Monopolar Stimulating Instrument is indicated for stimulation of cranial and peripheral motor nerves for location and identification during surgery, including spinal nerve roots. The RhythmLink International Monopolar Stimulating Instrument is sterile and for single use only.

Caution

Monopolar Stimulating Probes are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product. Product is sterile only if packaging is unopened. Do not use near high energy radio frequencies.

Packaging

- [1] 1.5mm Monopolar Stimulating Probe
- [1] Removable handle for use with 1.5mm probe
- [1] 2.5m cable with touchproof connector on both ends

Instructions for Use

Insert either end of the cable assembly into the slot provided on the handle so the cable leads away from the handle. Slide the proximal end of the MSP instrument into the opposite end of the handle until it is fully engaged with the connector assembly. Connect the opposite end of the cable assembly into the monitoring instrumentation for stimulation. Do not bend Probe. This product is single-patient use only. Discard Probe after use.

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- Reuse of this device may cause cross contamination and has NOT been validated for multiple uses.
- Re-sterilization of this device may cause unknown malfunctions. EO re-sterilization has not been validated for this device and may contribute to excessive EO gas residuals.

To view a list of symbol definitions found on packaging and instructions for use, please visit RhythmLink.com/symbols.

RhythmLink® is a registered trademark of RhythmLink International, LLC.
L178001 rev005

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Mdi Europa GmbH