



**Rhythmlink International, LLC** 866.633.3754 (toll-free)  
 1140 First Street South 803.252.1222  
 Columbia, SC 29209 803.252.1111 (fax)  
[Rhythmlink.com](http://Rhythmlink.com) [sales@rhythmlink.com](mailto:sales@rhythmlink.com)

## MR Conditional\*/CT Webb™ Electrode\*\*

### Materials

- Electrode - ABS Plastic Coated with Ag/AgCl
- Leadwire - Tinned Copper with PVC Jacket
- Connector - DIN 42 802
- Sterilized - No

### Product Specifications

- Plating - Ag/AgCl
- Electrode Size - 10 mm
- MR Unsafe Leadwire Extensions - 1.0, 1.5 and 2.5 m
- MR Conditional/CT 240 mm disconnect

### Environment

- Latex-Free Electrode & Leadwire
- PVC-Free Packaging

### Packaging

- Pouch - Polyethylene
- Pouch Size - 121 x 195 mm

### Additional Benefits

- Flat design to increase patient comfort
- Larger recording surface in comparison to traditional cups
- 4 color groups, up to 48 electrodes
- Numbered heat shrink to ensure accuracy upon reconnection

Rhythmlink's Disposable MRI Conditional\*/CT Webb Electrodes are intended for use in the recording of the EEG, EP or as a Ground or Reference in an EEG or EP recording. Eliminate the need to remove and reapply EEG electrodes each time a patient requires imaging. These electrodes are non-sterile and may remain on the patient each time they require a MRI. The Disposable MRI Conditional/CT Webb Electrodes cannot be used for data collection while in a MR environment because the extension cables must be removed prior to entry into the MR environment. After leaving the MR environment, use the numbered heat shrink to reconnect the extension cables to the matching electrode.

Visit [Rhythmlink.com/MR-CT](http://Rhythmlink.com/MR-CT) for additional information.

**Directions for Use** - Clean application site. Apply Cup Electrode using Weaver Ten20® conductive paste. MRI Conditional/CT Disposable Webb Electrodes are only approved for use with Weaver Ten20 conductive paste. **Remove all extension cables before entering a MR environment** and reconnect to resume data collection following the completion of the MRI. When the entire data collection session is complete, remove electrodes and clean patient application sites according to individual protocol.

**MRI Safety Information** - Non-clinical testing has demonstrated that the MRI Conditional Disposable Webbed Electrode is MR Conditional in configurations of 2 to 48 electrodes. These electrodes can safely remain on a patient during a MR scan for 15 minutes under the following conditions:

- Static magnetic field of 1.5 or 3.0 Tesla
- Maximum spatial gradient field of 4,000 gauss/cm [40T/m] or less
- Maximum whole-body averaged specific absorption rate of 2 W/kg in the Normal Operating Mode
- **Remove extension cables before entering an MR environment. They are MR Unsafe**

**Cleaning & Disinfecting Instructions** - This product is single-patient use only. Discard Electrode after use.



\*Patent Pending  
 \*\* Received FDA 510(k) Clearance [K130287] on May 30, 2013  
 \*\* Patent NO. D 644, 738