English

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## Disposable Subdermal Needle Electrode

### Intended Applications

RhythmLink International Subdermal Needles are intended for use with recording, monitoring and stimulating equipment for the purpose of recording of biopotentials such as Electromyography (EMG), Electroencephalography (EEG) and Nerve Potential signals.

### Caution

Subdermal Needles are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product. Product is sterile only if packaging is unopened. When selecting a Subdermal Needle Electrode, it is important to choose the appropriate length and diameter.

### Instructions for Use

Select appropriate Subdermal Needle Electrode. Prior to use, remove Needle sheath. When introducing Electrode into patient, do not insert up to hub. If needle bends before, during or after insertion, do not straighten or re-insert. Discard and replace with a new Electrode. This product is single-patient use only and should not be sterilized or reused. Discard Electrode after use.

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STERILE EO

EC REP

### European Authorized Representative

Mdi Europa GmbH  
Langenhagener Str.71  
D-30855 Hannover-Langenhagen

- Reuse of this device may cause cross contamination and has NOT been validated for multiple uses.
- Re-sterilization of this device may cause unknown malfunctions. EO re-sterilization has not been validated for this device and may contribute to excessive EO gas residuals.

To view a list of symbol definitions found on packaging and instructions for use, please visit [RhythmLink.com/symbols](http://RhythmLink.com/symbols).

RhythmLink® is a registered trademark of RhythmLink International, LLC.

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RhythmLink® é uma marca comercial registrada da RhythmLink International, LLC.