



English

## EMG Disposable Needle Electrode

### Intended Use

Rhythmink's Disposable EMG needle electrode is part of a system for detecting and recording bioelectrical activity of muscles for the diagnosis of functional impairment resulting from nerve injuries/disorders or from myogenic diseases as part of electromyography (EMG) examinations. This product is not intended for use as a stimulating needle.

### Caution

This product is for professional use only and should only be used in compliance with accepted industry standards. Rhythmink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product. Product is sterile only if packaging is unopened. Federal law restricts this device to sale by or on the order of a physician.

### Instructions for Use

Select appropriate needle electrode. Prior to use, attach device to extension cable (if applicable) and remove needle sheath. When introducing electrode into patient, do not insert up to the hub. If needle bends before, during or after insertion, do not straighten or re-insert. Discard and replace with a new electrode. This product is single-patient use only and should not be resterilized or reused. After use, treat device as hazardous waste material and dispose of accordingly. To avoid injury, do not attempt to reinsert used needle into protective sheath.



Rhythmink International, LLC  
1140 First Street South  
Columbia, SC, USA 29209-3540  
+1.866.633.3754 [toll-free]  
+1.803.252.1222  
+1.803.252.1111 [fax]  
sales@rhythmink.com  
Rhythmink.com

STERILE R

Pre-Sterilized Disposable

Rx Only



- Reuse of this device may cause cross contamination and has NOT been validated for multiple uses.
- Re-sterilization of this device may cause unknown malfunctions and has not been validated for this device.
- The device is not made with natural rubber latex.

To view a list of symbol definitions found on packaging and instructions for use, please visit [Rhythmink.com/symbols](http://Rhythmink.com/symbols)

Rhythmink® is a registered trademark of Rhythmink International, LLC.